



Exporting Images as JPEG/TIFF for Conference or Publications


1. Find image you want in GE PACS.
2. Click drop down arrow under Exam at the top

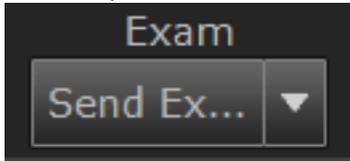

3. Click on 'Send Exams'

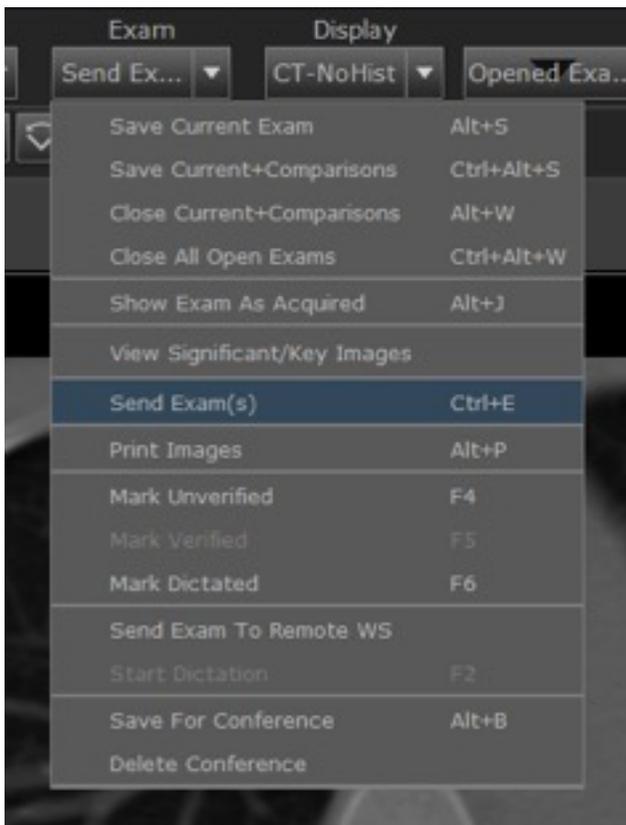

4. Click on Export Images tab

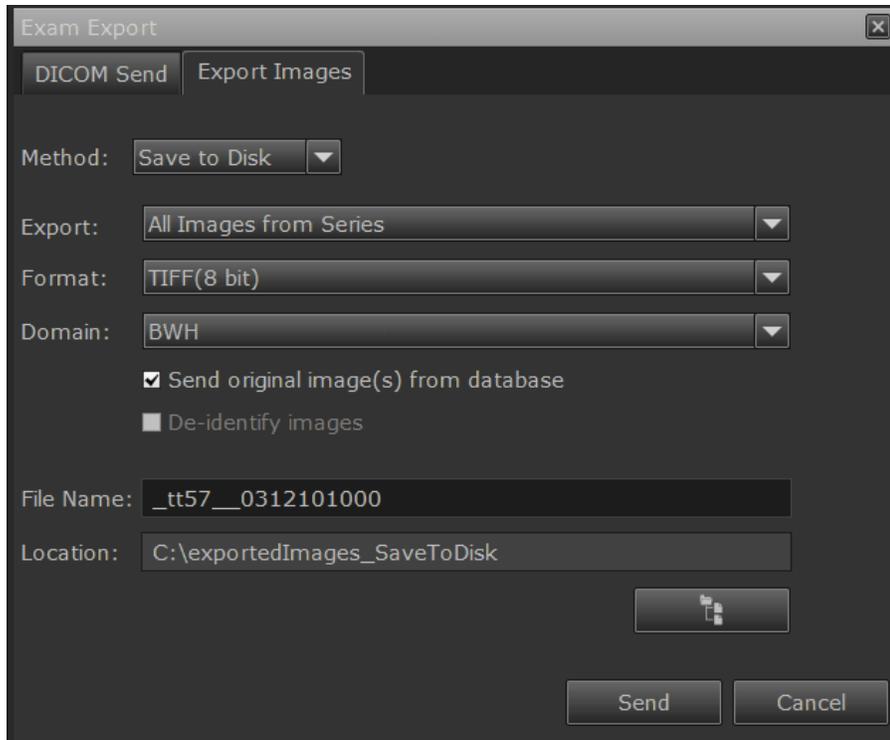

5. Fields:

- a. Method: Save to Disk
- b. Export: 'All Images from Series' is self explanatory, 'significant images' are the ones you select using this tool  where the letters 'SI' will appear on the image, or 'selected image' which is the image you are currently on when you click send exams.
- c. Domain: Keep BWH
- d. File Name: self explanatory
- e. Location: change destination of the exported images by clicking on the directory

button .

6. Click send to export.