


### Transferring TAVR images from the Toshiba

- 1) Select "Raw Data" to find the patient list
- 2) Click once to highlight your patient from the list
- 3) Click "Load"

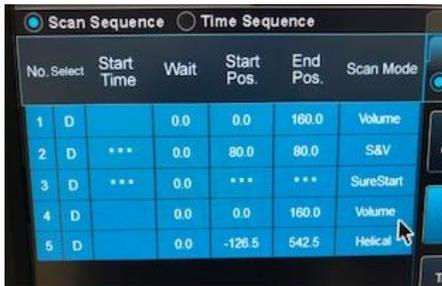

- 4) Select the second "Volume" series
- 5) Click "ECG"



- 6) Check to make sure all red dots are in the highlighted heart beat
- 7) If there are red dots outside, you must click them to remove them
  - a. You can also move the red dots to adjust the range of the data



- 8) Click "CFA"
- 9) Make the range from "0" to "95" with step of "5"



10) Click "Transfer"

11) Select "APPISERVER" and "GE PACS"



12) Highlight the text in "D-FOV" (Field of view)

13) Change to "180"

14) Ensure the smaller field of view completely surrounds the heart by scrolling through the entire area and dragging the yellow circle until it's correctly aligned



15) Select the text in "Series Description" to rename the new series

16) Double check everything looks okay

17) Click "Recon"

